Basic 3 Litri/Basic 3 Liters

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Basic 3 Litri/Basic 3 Liters

The ultrasonic tub is designed to wash instruments used in the medical, dental, cosmetic, veterinary fields of application and all those areas which require an ultrasonic washing. The cleaning device involving ultrasound (via cavitation) allows you to thoroughly clean places otherwise difficult to reach by manual cleaning, without any risk resulting from unsafe handling of contaminated instruments.

Dati tecnici • Technical data	Basic 3
LxAxP / WxHxD	305x248x168 mm
LxAxPvasca / WxHxD vessel	235x100x135 mm
Peso / Weight	3,7 Kg
Potenza ultrasuoni Ultrasonic power	150 W

Characteristics:

- Standard tank (3 ltrs.)
Perfect for small rooms
- Standard tank (3 ltrs.)
Perfect for small loads
- 1 Transducer
High cleaning power
- Sweep Mode
High cleaning power

Degassing (DEGAS)

The device starts activating the ultrasonic phases intermittently. This process reduces the gas dissolved in the liquid, thus improving the efficiency of the subsequent washing.

Ultrasonic cleaning

Washing phase through ultrasonic cavitation.

SWEEP

Ultrasounds are modulated in frequency, thus reducing the negative effects of standing waves, increasing the distribution of ultrasonic energy

within the washing liquid. The cleaning is more efficient (improved cavitation).

Discharge

In the version without discharge (BASIC 3), the tub is to be emptied manually.