Basic 9 Litri/Basic 9 Liters

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Basic 9 Litri/Basic 9 Liters

The ultrasonic tub is designed to wash instruments used in the medical, dental, cosmetic, veterinary fields of application and all those areas which require an ultrasonic washing. The cleaning device involving ultrasound (via cavitation) allows you to thoroughly clean places otherwise difficult to reach by manual cleaning, without any risk resulting from unsafe handling of contaminated instruments.

Dati tecnici • Technical data

Basic 9

LxAxP / WxHxD	418x268x259 mm
LxAxPvasca / WxHxD vessel	332x120x227 mm
Peso / Weight	7,1 Kg
Potenza ultrasuoni / Ultrasonic power	300 W

Characteristics:

- Large capacity tank
- Variable loads management
- Large capacity tank
- Surgical trays placement
- 4 Transducer
- High cleaning power
- Sweep Mode
- High cleaning power

Degassing (DEGAS)

The device starts activating the ultrasonic phases intermittently. This process reduces the gas dissolved in the liquid, thus improving the efficiency of the subsequent washing.

Ultrasonic cleaning

Washing phase through ultrasonic cavitation.

SWEEP

Ultrasounds are modulated in frequency, thus reducing the negative

effects of standing waves, increasing the distribution of ultrasonic energy within the washing liquid. The cleaning is more efficient (improved cavitation).

Discharge

In the version without discharge (BASIC 9), the tub is to be emptied manually.