# Free 3 Litri/Free 3 Liters

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# Free 3 Litri/Free 3 Liters

The ultrasonic tub is designed to wash instruments used in the medical, dental, cosmetic, veterinary fields of application and all those areas which require an ultrasonic washing. The cleaning device involving ultrasound (via cavitation) allows you to thoroughly clean places otherwise difficult to reach by manual cleaning, without any risk resulting from unsafe handling of contaminated instruments.

Dati tecnici • Technical data		Free 3
LxAxP / WxHxD		305x248x178 mm
LxAxPvasca / WxHxD vessel		235x100x135 mm
Peso / Weight		4 Kg
Potenza ultrasuoni Ultrasonic power		150 W

### Characteristics:

- Standard tank (3 ltrs.)  
Perfect for small rooms
- Standard tank (3 ltrs.)  
Perfect for small loads
- 1 Transducer  
High cleaning power
- Sweep Mode  
High cleaning power
- Easy to manage  
Discharging tap

### Heating

The device proceeds to the heating phase of the detergent. It stops automatically when the preset temperature is reached. Always consider that during the ultrasonic cleaning the temperature of the detergent tends to rise independently.

### Degassing (DEGAS)

The device starts activating the ultrasonic phases intermittently. This process reduces the gas dissolved in the liquid, thus improving the efficiency of the subsequent washing.

### Ultrasonic cleaning

Washing phase through ultrasonic cavitation.

### SWEEP

Ultrasounds are modulated in frequency, thus reducing the negative effects of standing waves, increasing the distribution of ultrasonic energy within the washing liquid. The cleaning is more efficient (improved cavitation).

### Discharge

In the optional self-discharge version (FREE 3), the tub is emptied by opening the appropriate valve.