Visore GVS32

GVS32 viewer

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Visore GVS32

GVS32 viewer

Protective against biological agents and cytotoxic drugs

Description:

GVS32 protective viewer, thanks to its physical characteristics grants a right protection to the ocular area, when risky situations are expected.

This viewer is CE marked as PPE in II Category in compliance with EC Directive 686/89.

Viewer is in compliance with European Regulation EN 166:2001.

It is suitable also to defend workers exposed to droplets and liquids containing biological agents and cytotoxics.

It can also be used by people wearing personal glasses and it fits different head sizes, thanks to its supporting elastic stripes.

Special Technical features:

Weight: 74 gr.

Soft anti allergenic foam touching the forehead

Interchangeable viewer

Anti misting treatment

Visual field more than 220°

Cold - sterilized

Application/areas of use

Suitable for the workers' protection against droplets and liquids containing biological agents.

Areas of use are:

Hospital settings	Health services/First Aid
Veterinary	Industrial setting
Pharmaceutical	Spray painting
Technologic	Industry White room
Police-crimeview	Food processing industry
Rubbish collecting services	Agriculture

Certification

Marking on frame: C EN 166 3 CE

Marking on lenses: C 1 S3 N CE

Model:

DESCRIPTION

Protective viewer against biological risk GVS32

CODE

SFV00101

WEIGHT[g]

74

DIMENSION [cm]

22 x 22 x 10

Visore GVS32

GVS32 viewer

FRAME MARKING

	C	EN166	3	CE
Cl.Com Advanced Technology ID	----- ----- -----			
European Norm compliance	-----			
Droplets and sprays of liquids protection	-----			
CE Marking	-----			
Size of the marking in mm	5	5	5	

Position of the marking The marking is on the upper right side of the lens *

VISOR MARKING

	C	1	S	N	CE
Cl.Com Advanced Technology ID	----- ----- ----- -----				
Optical Class	-----				
Improved particles impact resistance	-----				
Droplets and sprays resistance	-----				
Foggy resistance of the visor	-----				
CE Marking	-----				

Position of the marking The marking is on the upper right side of the visor *

*The right and left position are intended once the viewer is on.

The substitution of the damaged visor with an original spare one, having the same size, must be done by skilled people, as follows:

- keep the viewer turned upside
- unscrew the tightening supports positioned at the temples and remove the damaged visor
- put the spare visor on the same place
- place the tightening devices and screw them up
- after the cleaning, check if any scratches or bruises avoid a right vision.

To do that, the operator must have clean hands or wear non abrasive gloves. The viewer must be washed and dried attentively before and after the replacement.

If the above mentioned cleaning instructions are followed, the GVS32 viewer will not deteriorate, when used normally. The lenses must be replaced whenever permanent damages as stretches or cuts occur. The equipment must be also replaced when a strong impact occurs

Materials used (latex free & solvent free)

- Frame: anti allergic elastic band, soft and anti allergic foam touching the forehead.
- Lenses: thermo moulded acetate with outside and inside no foggy treatment

Sterile

The PPE can be sterilized by ethylene oxide.

Visore GVS32

GVS32 viewer

Storage

The PPE must be stored correctly in their original packaging and kept away from heat sources.

The shelf life of the PPE in their original packaging, if kept correctly storage inside their packaging, it is 5 years .

The correct storage conditions are: room temperature from + 5 °C to + 40 °C ; R.H. < 60 %

Expire date

5 years from production date, following the above mentioned storage conditions in the original packaging.

Instruction for use

Please read the instruction before use

Application limits

The GVS32 viewer has been studied to grant a right protection to the ocular area when there are possible risks of impact at low energy by small metallic particles moving not faster than 5 m/s and grant the workers' protection against droplets and liquids containing biological agents.

It is not allowed to use for other applications.

The material getting in touch with the user skin could cause allergic reactions to very sensitive subjects

Utilization

In case of exposition to biological agents it must be used just once and it cannot be washed or disinfected to used again.

For other uses, if the cleaning instructions are followed, the CL 32 viewer will not deteriorate, when used normally.

The lenses must be replaced whenever permanent damages as stretches or cuts occur.

The equipment must be also replaced when a strong impact occurs